

Honeywell N95 Flatfold Disposable Respirators, 20 Face Mask Pack - DF300N95BX





slide 1 to 6 of 7



[Video](#)



Availability: In Stock

Price: **\$25.95** ~~\$37.95~~

You Save: **\$12.00** (31% Off)
[- 773 Customer Reviews](#)

1 Add to Cart

Product Features

[Buy In Bulk](#) [Videos](#)

[Description](#) [What is an N95 Face Mask?](#)

NIOSH approved DF300 Series Flatfold Disposable Respirators are made with a multilayered absorption media that repels moisture and features a smooth inner lining for added comfort. Honeywell NIOSH-Approved N95 respirator face masks provide protection against airborne particulates with a 95% filter efficiency against solid and liquid particulates in the air (excluding oils). Each package includes (twenty) disposable N95 respirators.

Product Details

- Standards : NIOSH Approved
- Filtration Type : N95
- Mask Type : Flatfold Molded Cup
- Latex Free : Yes



USE INSTRUCTIONS

1. Failure to follow all instructions and limitations concerning the use of the respirator and/or failure to wear the respirator for the whole duration of exposure to contaminants can seriously reduce the performance characteristics of the respirator and lead to illness, injury or death.
2. Before Occupational Use of this respirator, a written respiratory protection program must be implemented meeting all the local government requirements. In United States, employers must comply with OSHA 29 CFR 1910.134 which includes medical evaluation, training, and fit testing.
3. Discard the respirator and replace with a new one if:
 - Excessive clogging of the respirator causes breathing difficulty.
 - The respirator becomes damaged.
4. Visually inspect the filtering facepiece prior to each use to assure that there are no holes in the breathing zone, especially around the weld attachment of the headbands to insure no damage has occurred. Discard respirator if any visual signs of holes or damage.
5. Leave the contaminated area if dizziness, irritation or other distress occurs.

USE LIMITATIONS

1. Do **NOT** use the respirator or enter or stay in a contaminated area under the following circumstances:
 - Atmosphere contains less than 19.5% oxygen.
 - Atmosphere contains oil aerosols.
 - For protection against gases or vapors.
 - Contaminants or their concentrations are unknown or immediately dangerous to life or health.
 - Concentrations or contaminants exceed maximum use concentrations in applicable OSHA standards or applicable government regulations or 10 times the PEL (Permissible Exposure Limit), whichever is lower.
 - For sandblasting, paint-spray operations, asbestos.
2. Do **NOT** modify or misuse the respirator.
3. Do **NOT** use the respirator with beards or other facial hair that interferes with direct contact between the face and the edge of the respirator, or any other conditions that may prevent a good face-seal.

FITTING INSTRUCTIONS

Follow those instructions each time respirator is worn.

1. Cup the respirator in the hand, allowing the head straps to hang freely.
2. Hold the respirator under the chin with the nose-piece facing outwards. Place the lower head strap around the neck as seen in picture 2 above.
3. Holding the respirator against the face with one hand, place the top head strap above the ears, around the crown of the head.
4. Place both fingers on the nose clip and apply pressure onto the nose clip, forming it to the shape of your nose.
5. Place hands on each side of the respirator and move slightly right left, up and down to adjust the position of the respirator and achieve the most optimal fit on the face.

FIT-CHECK THE FACE-SEAL AS FOLLOWS

1. Place both hands over the respirator - do not disturb the position of the respirator;
2. Inhale sharply (a negative pressure should be felt inside the respirator);
3. If you detect air-leaks, readjust the head straps and/or the nose-piece.

⚠ WARNING:

If you cannot achieve satisfactory fit, do **NOT** enter the contaminated area. See your supervisor.

Product Specs

UPC: 883940393025

MPN: DF300N95BX

Category: [Facemasks and Dust Masks](#)

[Product Video](#)

[Buy In Bulk](#)

Customers Also Bought



[Quick view](#)

[Honeywell Saf-T-Fit Plus N95 Disposable Respirator, 20-Pack](#)

MPN: RWS-54003 (80)

\$25.95 Save: \$13.00



[Quick view](#)